

# The Karadent™ Advantage



Semi-rigid microcrystalline polymer for full dentures, semi-flexible partial, clear clasps, frameworks and nightguards

Highly aesthetic  
BPA and monomer free  
Maximum strength and durability  
No volume shrinkage-exceptional fit  
Minimal water absorption  
Quick and easy chairside adjustments  
Easy to repair, rebase, and reline with acrylic



Clear Framework



Nightguard



Pittman Dental Laboratory is making TCS Karadent™ a top choice for removable appliances.

## Insurance Codes

D5225 Maxillary Partial Denture

D5526 Mandibular Partial Denture

D5110 Maxillary Complete Denture

D5120 Mandibular Complete Denture

## Benefits

Economical, aesthetic solution for missing teeth Metal-free

## Tips

- 1) For best results use lightweight alginate impression materials.
- 2) Adjusting Karadent™ is quick and easy. You can adjust using the same techniques as acrylic dentures.
- 3) Use a carbide bur for making small adjustments and Green Mounted Stones to smooth the area.
- 4) For a touch-up polish, Blue Hi-Shine polishing paste (item #4215-01) is recommended.
- 5) Recommend TCS Dental Appliance Cleaner to your patients for optimal maintenance.

## Shades

4 options: Standard, Medium and Dark Pink (Mahari) and Clear

*Combines the unbreakable strength of nylons with the simplicity of acrylics to create what technicians and dentists have been waiting for in removable appliances!*

