

Introducing



YOUR SMILE
— FACTOR



Clear Aligner System

It's time to add a new offering to your practice, "Your Smile Factor" clear aligners. The clear aligner market is growing by leaps and bounds with online and overnight startups, but many consumers still want the direction and guidance of their doctor during this process.

Pittman Dental Laboratory has been working with doctors for over 45 years and has the experience in delivering quality dental products and services to our customers.



There are four types of alignment trays available in both single and dual aligners:

Description	Trays Included
Specific Design Aligners	1 to 3
Partial Aligners	4 to 6
Short Term Aligners	7 to 11
Full Aligners	12 Plus

To learn more visit:
www.pittmandental.com/clear-aligners-doctors




2355 Centennial Circle, Gainesville, GA 30504
800-235-4720 | www.pittmandental.com
Support@pittmandental.com



The Your Smile Factor Process How to order clear aligners:



Take a physical mold of the patients teeth or take a digital scan and upload it through the Doctor Portal on our website. Upload pictures of your patients teeth through the Doctor Portal. Your Smile Factor technicians will verify molds and digital files and provide a computer graphic representation of the projected teeth movements, created in the Your Smile Factor software program. This verification process is then provided to the doctor and patient for approval or modification before aligners are manufactured.

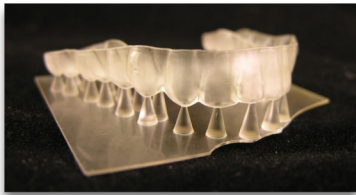

The aligners are modeled using CAD-CAM software and manufactured using a rapid prototyping technique called stereolithography.

The molds for the aligners are built in layers using a photo-sensitive liquid resin that cures into a hard plastic when exposed to a laser. The aligners are made from an elastic thermoplastic material that applies pressure to the teeth to move into the aligner's position.

Once the doctor has approved the models, the aligners will be shipped.



Contact Us Today!

2355 Centennial Circle, Gainesville, GA 30504
800-235-4720 | www.pittmandental.com
Support@pittmandental.com